



Cleaning and sterilizing advices

Caution! The implant drills are not delivered sterilized. Sterilize before use.


Recommendations

The martensitic stainless steel does not completely resist to corrosion; below some advices:

- **Caution: Avoid any contact between the plastic box and chemical products (solvent, chlorinated hydrocarbons) or cleaning solution or disinfectant containing ammonia. These may cause deterioration of the plastic box.**
- Do not use cleaning solution or disinfectant containing chloride or acid (caution with the tap water too much chlorinated).
- Avoid solution with aldehyde, owing to the ability of this product to keep the proteins.
- Eliminate meticulously all the post-operative residues (blood, bone ...) on the drills, or inside for the drills with internal irrigation. Use a nylon brush.
- Avoid, as much as possible, shocks and contacts with other drills or instruments (damaging of the quality surface and / or cutting power)
- Do not clean, in the same ultrasonic box, the implant drills with other tools which material is different.
- Use suitable cleaning solutions. Respect the manufacturer directions for use.
- The organic matter must be removed from the box in ultrasonic vats, using enzymatic detergent (three enzymes minimum) in dilution recommended by the manufacturer. After washing it in running water, search for possible residuals and repeat the process. If necessary remove residuals by manual cleaning.
- Rinse with distilled water, and then dry all the parts immediately after the cleaning process. It is necessary to be meticulous during the drying.
- Stock the drills in clean and dry areas.
- Sterilization by wet heat autoclaving 134°C (273° F) – 18 minutes. Make sure that the elements, inside the autoclave, are not rusted.
- **Do not let the box touch the autoclave inner wall.**
- For the drills with removable stops: these steps must be carried out individually on the dismantled component.
- The implant drills cannot be used indefinitely.

WAM

Tech'Indus - B21 - 645, rue Mayor de Montricher
13854 Aix-en-Provence Cedex 3 - FRANCE
Tel: +(33) 4 13 41 57 80

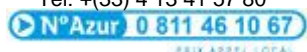

Fax: +(33) 4 88 71 43 11

Email: plusinfo@wamkey.com

SARL au capital de 7650 Euros – RCS Aix-en-Provence
433 231 461 000 40 – TVA: FR28433231461

1910 Salaberry
Montréal (QC)
H3M 1K4 - CANADA
Tel: +(1) 514 830 6245 (local)
Tel: 1-888-369-1448 (Toll Free / Sans Frais Usa/Canada)
Email: plusinfo@wamkey.com
NEQ 1166532409