INSTRUCTIONS FOR USE
STAINLESS PROPHYLAXIS STRIP
ACERO INOXIDABLE CINTA PROFILÁTICA
AÇO INOX CINTA PROFILÁTICA

CE
MS: 80676920066

IFU - Rev. 00
03/May/2023
Page 1 of 6

ENGLISH

Indication:

Prophylactic brace is indicated for clinical use in the removal of bacterial plaque and calculus of the interproximal region.

Fundamentals of its operation and action:

Prophylactic brace is essential in prophylaxis work, as it easily accesses the proximal region. Its active face acts effectively in the removal of dental stones. The circular microblades, located around the holes in one of the faces of the tape, have as their main characteristic an action against the stones, preserving the integrity of the enamel and the restorative materials.

Instructions for use:

Insert the strap in the interproximal space, noting that the active face is facing the desired proximal one.

Hold the tape by the ends, pull it against the tooth and apply it in longitudinal movements in the form of back-and-forth until the desired effect is obtained. Remove the tape.

Contraindications:

- Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and should not be used.
- Do not use cold sterilization methods (chemical process) to sterilize galvanized steel abrasive strips. These chemical agents can corrode the products.

Warnings and Precautions:

- Professional use.
- Single use only.
- Its reuse can cause cross contamination.
- When there is a proven allergy to its component, the product should not be used.
- The contact of the tape with the gum can cause injury to the patient.
- It is recommended to protect the papilla.
- Reuse may affect product performance.
- Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State where the users and/or patients are established.

Conservation and storage:

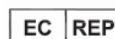
- The product must be stored in a ventilated, dry place protected from light, to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.
- Keep the product in its original packaging until use. Inspect packaging before use.
- The operator must be informed of the content. Precautions to be taken during transportation are shown on the packaging, ensuring the protection of the instrument from shipment to delivery to the customer.
- Packages should not be left out in the open.

Other additional information:

- If necessary, 3R provides instructions for use in printed format, at no additional cost, upon request by email (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will get back to you within 7 business days of the request.
- Always check the website: www.microodont.com.br for the most up-to-date version of the instructions for use corresponding to the product.

Composition: Stainless Steel and Aluminum Oxide.

Symbology:

INSTRUCTIONS FOR USE
STAINLESS PROPHYLAXIS STRIP
ACERO INOXIDABLE CINTA PROFILÁTICA
AÇO INOX CINTA PROFILÁTICA

CE
MS: 80676920066

Symbol	Symbol Title	Description
	Expiration / Use-by date	Indicates the date after which the health product should not be used.
	Consult instructions for use	Indicates the user's need to refer to the instructions for use.
	Date of manufacture	Indicates the date when the health product was manufactured
	CE marking	Indicates that the product has CE marking and the identification of the Notified Body. (When applicable)
	Keep away from sunlight	Indicates a health product that needs protection against light sources.
	Manufacturer	Indicates the Manufacturer of the product for the health as defined in the Directive of the European Community 90/385EEC, 93/42/EEC and 98/79/EC.
	Batch Code	Indicates the manufacturer's batch code so that the batch or departure can be identified.
	Catalog number	Indicates the manufacturer's reference number so that the product for the health can be identified.
	Non-Sterile	Indicates that the product is not sterile
	Recyclable	Indicates that packaging can be recycled
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	Authorized representative in the European Community	Indicates the Authorized representative in the European Community

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