

Purpose of use

Solid Gutta Percha used for root canal filling

Preparation before use

1. Before using the product, check the expiration date and make sure there is no dirt, dust, or any foreign substance. It is strongly recommended not to use the product in case it has been hardened.
2. The user has to read and fully understand the manufacturer's instructions and then use the product.
3. Prepare all accessories and instruments required prior to the procedure.
4. Use only within the validity period of the product.

Directions for use

- 1) In order to fill a root canal, a dental professional must follow the following steps.

◆ in case of a Gutta Percha Obturator

- ① Insert a Gutta Percha Obturator to a backfill obturation device and soften it by heat.
- ② Inject the soften and melted Gutta percha obturator into a root canal slowly and then press and compact the Gutta percha obturator by plugger instrument.
- ③ The rest of parts should be filled in the same way.
- ④ Take an X-ray after finishing the filling procedure to ensure the complete filling of the canal.

◆ in case of a Gutta Percha Cartridges

- ① First check the device whether it can be used with Gutta percha Cartridges.
- ② Insert a Gutta Percha Cartridge to a backfill obturation device and soften it by heat.
- ③ When the Gutta percha cartridge softened by a heater of root canal filling device, press the operation button and check if the melted Gutta percha obturator comes out through the tip. After that, inject the melted Gutta percha obturator into a root canal slowly and then press and compact the Gutta percha obturator by plugger instrument.
- ④ The rest of parts should be filled in the same way.
- ⑤ Take an X-ray after finishing the filling procedure to ensure the complete filling of the canal.

The procedure of operations listed above is a basic and essential way, according to the judgment of the dental professional or specialist, it might be changed by other way of treatment method or instrument, device used. The user can determine the procedure of operation as appropriate according to the symptoms of the patient.

Maintenance after use

1. The remaining Gutta percha obturator has to be removed from the device and disposed.
2. After use, close the lid to prevent mixing of dust and other debris and store at room temperature.
3. Do not re-use. It is single use.

Storage

1. Store at room temperature.
2. Keep away from flammable and combustible materials and heat.
3. Do not toss or throw the products while conveying.
4. Store separate from other products of different sizes and types.

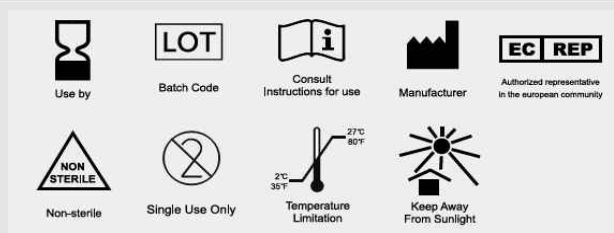
Precautions for use

◆ General caution.

1. It must be used by expert, dental professional only. The directions and precaution for use has to be complied strictly.
2. Do not use the patient with allergy, inflammation or infection to this product.
3. This product can not be prescribed in doses.
4. Keep in sealed after use.

◆ Caution for applying

1. Before operation, be sure to use a rubber dam to protect the airway.
2. Before root canal filling, check the secondary infection inside the canal.
3. After finishing the filling procedure, check the tightness by taking an X-ray

Symbols**Warranty and Limitation of Liability**

DiaDent Group International will replace product that is proven to be defective. DiaDent Group International does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described.

Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

European Representative

DiaDent Europe B.V.
 Antennestraat 70, 1322AS
 Almere, the Netherlands
 Tel: 00 31 (0)36 549 8607 FAX: 00 31 (0)36 536 7317

Contact Information**DiaDent Group International**

16, Osongsaengmyeong 4-ro, Osong-eup,
 Heungdeok-gu, Chungju-si,
 Chungcheongbuk-do, 28161, Rep. of Korea
 Tel : 82-43-266-2315
 Fax : 82-43-262-8658
<http://www.diadent.co.kr>
 E-mail: diadent@chollian.net

DiaDent Group International Inc.

11-3871 North Fraser Way, Burnaby
 B.C., Canada, V5J 5G6
 Tel : 1-604-451-8851
 Fax : 1-604-451-8865
www.diadent.com
diadent@diadent.com

DiaDent Europe B.V.

Antennestraat 70, 1322AS
 Almere, the Netherlands
 Tel : 31-36-549-8607
 Fax : 31-36-536-7317
diadentneurope.com
diadent@diadentneurope.com