INSTRUCTIONS FOR USE
TAMBOREL
FILE DISPENSER
PORTA LIMAS

IFU -Tamborel - Rev. 00
03/May/2023
Page 1 of 4

ENGLISH

Indication:

The tamborel is used as support for endodontic instruments.

Fundamentals of its operation and action:

The tambourine was developed as a support to accommodate endodontic instruments (files) during endodontic treatment.

Instructions for Use:

Open the lid of the file door, between the lid and the base put a sterile gauze and close, insert the files or other instruments in the gauze so that it is retained, helping the organization of the workbench, avoiding damage to the active part of the instrument.

Cleaning procedure:

• After use the device should be washed with neutral detergent (enzymatic detergent) manually or in an ultrasonic vat, with a brush other than steel, so as not to create microcracks and retain microorganisms, perform drying with paper towels or compressed air jet, pack and sterilize in autoclave at 135°C for 10 minutes.

Contraindications:

Not applicable

Warnings and Precautions:

- The product must be sterilized in autoclave up to 135°C before use.
- The cleaning of the Tamborel can be done with a proper brush for cleaning, water and neutral soap (enzymatic detergent). Clean and dry material should follow autoclave sterilization procedures.
- The material must be packed in surgical grade paper for sterilization.
- LOT Number: It is important to keep the batch number, identified on the product label, to be informed in any occurrences with the manufacturer.
- The disposal of the material must follow the relevant local legislation.

Conservation and storage:

- Store in a clean, dry place.
- Transport in a clean, dry and room temperature location.

Other additional information:

- If necessary, 3R provides the instructions for use in printed format, at no additional cost, upon request by email (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will return within a maximum of 7 working days from the request.
- Always check on the website: www.microdont.com.br, the most up-to-date version of the instructions for use corresponding to the product.

Composition:

Drum composition: Aluminum 6351

Symbology:

Symbol	Symbol Title	Description
	Expiration / Use-by date	Indicates the date after which the health product should not be used.
	Consult instructions for use	Indicates the user's need to refer to the instructions for use.
	Date of manufacture	Indicates the date when the health product was manufactured
	Manufacturer	Indicates the Manufacturer of the product for the health as defined in the Directive of the European Community 90/385/EEC, 93/42/EEC and 98/79/EC.
	Batch Code	Indicates the manufacturer's batch code so that the batch or departure can be identified.
	Catalog number	Indicates the manufacturer's reference number so that the product for the health can be identified.
	Sterilized using irradiation	Indicates that the product is sterilized
	Keep away from sunlight	Indicates a health product that needs protection against light sources.
	Non-Sterile	Indicates that the product is not sterile
	Recyclable	Indicates that packaging can be recycled
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	Authorized representative in the European Community	Indicates the Authorized representative in the European Community

This review cancels and replaces the previous revision.

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