


Dental Cart Installation manual

1. Product Information


Product Name: Dental Cart

Product model: TC-6

2. Dental Cart composition

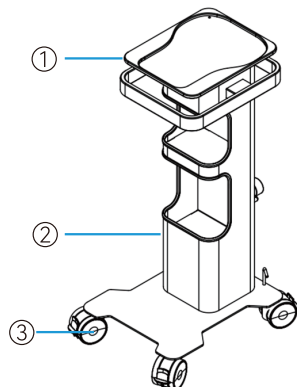

- ① Top plate;
- ② Column;
- ③ Bottom plate.

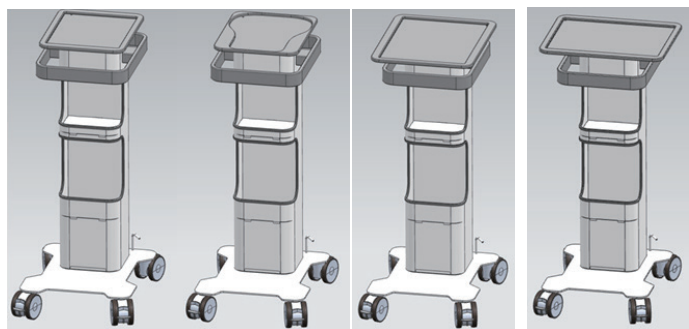


A Type

B Type


S Type

Versatile Type


3. Installation of small Cart

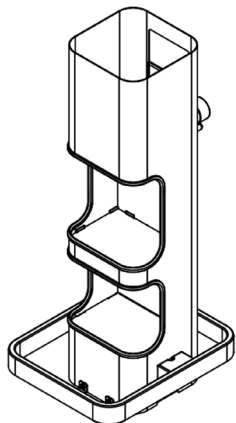


- Take out the small cart column and place it upside down.

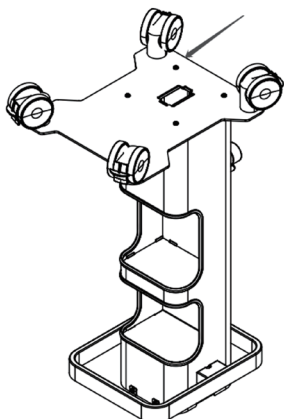


- Take out the bottom plate and install it on the pillar of the Cart. Use a screwdriver to tighten the bottom screw.

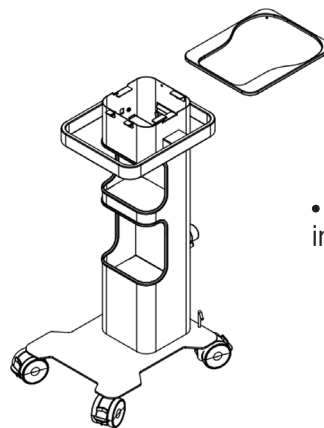

- Place the cart forward and install the top panel.

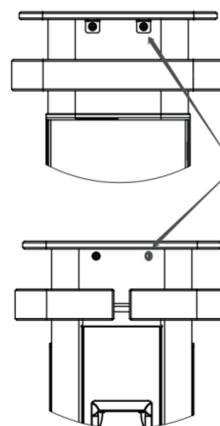

- Snap the top plate and column together, and tighten the front and rear fixing screws with a screwdriver.

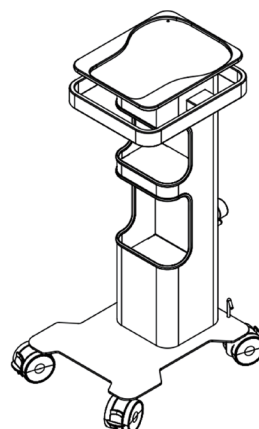

- Complete installation.

4. Storage conditions:

Dental carts should be stored indoors in a well ventilated, moisture-proof, and corrosion-free environment.



Guilin Woodpecker Medical Instrument Co., Ltd.
Information Industrial Park, Guilin National High-Tech
Zone, Guilin, Guangxi, 541004 P. R. China

Tel:

Sales Dept.: 0773-5873196/ 2350599

After-sales Service Dept.: 0773-5827898/13978361362

Website: <http://www.glwoodpecker.com>

E-mail: woodpecker4@glwoodpecker.com

ZMN-SM-832 V1.0-20230705