INSTRUCTIONS FOR USE
STAINLESS STEEL DIAMOND STRIPS
TIRAS DIAMANTADAS DE ACERO INOXIDABLE
TIRA DIAMANTADA DE AÇO GALVANIZADO

MS: 80676920013

ENGLISH

Indication:

Finishing and polishing interproximal surfaces

Fundamentals of its operation and action:

Abrasion

Instructions for use:

Abrasive stainless steel sandpaper strips feature a neutral (non-abrasive) center that is used to insert the sandpaper into the proximal faces without wearing down the contact area.

- Hold the sandpaper with the pads of your thumb and index finger
- Insert the files with the neutral center in the interproximals of the teeth
- Make movements from the buccal to the palatal/lingual and vice versa, from the cervical region towards the occlusal surface and vice versa
- In order to facilitate insertion on the proximal face, a lubricant can be used with the sandpaper.

Manual cleaning procedure:

- Use a designated area specifically for cleaning.
- Rinse the strips under running water and keep them immersed in an enzymatic detergent solution bath, following the manufacturer's instructions.
- After soaking and keeping it soaked, brush well away from your body with a neutral cleaning agent (enzymatic detergent). Care must be taken to avoid spreading contaminants from the spray during brushing procedures. Do not use wire brushes.
- Special care must be taken to clean cracks and other hard-to-reach areas. Visually inspect to confirm debris removal. Repeat the cycle if necessary.
- Rinse the strip well under running water until it is visibly clean.
- Dry the device with clean compressed air or a paper towel.

Sterilization requirements:

In case of reuse, it is recommended to sterilize the product in a saturated pressure steam autoclave at 135°C for 10 minutes, properly packed in its own autoclave packaging, or the sterilization parameters must be followed according to the relevant policies/rules country where the device is being used. It is the full responsibility of the professional to keep the autoclave equipment up to date to ensure the correct sterilization of the product.

Contraindications:

- Cleaning agents with chlorine or chloride as an active ingredient are corrosive to stainless steel and should not be used.
- Do not use cold sterilization (chemical process) methods to sterilize galvanized steel abrasive strips. These chemical agents can corrode products.

Warnings and Precautions:

- Non-sterile product. 3R recommends single use.
- Only dental surgeons should use this product, as they have the scientific information necessary for its proper use.
- Safety glasses must be worn to protect against ejected particles.
- Wear PPE, including gloves, masks, safety glasses, and long-sleeved lab coats when handling the Strip, as with all other dental instruments. Wear protective eyewear on the patient as well.
- BATCH NUMBER: It is important to keep the batch number, identified in the product title, in order to be informed of eventual occurrences with the manufacturer.
- Disposal of material must follow relevant local legislation.

Conservation and storage:

- The product must be stored in an airy, dry place, protected from light, in order to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.
- Keep the product in its original packaging until use. Inspect the packaging before use.
- The operator must be informed of the content. The care to be taken in transport is shown on the packaging, thus ensuring the protection of the instrument from shipment to delivery to the customer. Packages must not be left to the action of time.

Other additional information:

- If necessary, 3R provides instructions for use in printed format, at no additional cost, upon request by email (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will get back to you within 7 business days of the request.
- Always check the website: www.microodont.com.br for the most up-to-date version of the instructions for use corresponding to the product.

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
Composition: Stainless steel and Diamond.

INSTRUCTIONS FOR USE
STAINLESS STEEL DIAMOND STRIPS
TIRAS DIAMANTADAS DE ACERO INOXIDABLE
TIRA DIAMANTADA DE AÇO GALVANIZADO

MS: 80676920013

Symbol	Symbol Title	Description
	Expiration / Use-by date	Indicates the date after which the health product should not be used.
	Consult instructions for use	Indicates the user's need to refer to the instructions for use.
	Date of manufacture	Indicates the date when the health product was manufactured
	Manufacturer	Indicates the Manufacturer of the product for the health as defined in the Directive of the European Community 90/385EED, 93/42/EEC and 98/79/EC.
	Batch Code	Indicates the manufacturer's batch code so that the batch or departure can be identified.
	Catalog number	Indicates the manufacturer's reference number so that the product for the health can be identified.
	Sterilized using irradiation	Indicates that the product is sterilized
	Non-Sterile	Indicates that the product is not sterile
	Recyclable	Indicates that packaging can be recycled
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	Authorized representative in the European Community	Indicates the Authorized representative in the European Community

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